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REF T-PODR

QTY: 1

CONTENTS: Support Binder (Belt), Pulley System.

INTENDED PURPOSE: Provides circumferential compression to the pelvis in patients with suspected pelvic fracture for pelvic stabilization which may reduce blood loss and pain.

Where applicable dispose of according to local regulations for biohazardous waste.

When used in X-Ray: Detachable X-Ray Detectable Label is included to identify that T-POD is in place on patient.

WARNING: Single use: Do not reuse, reprocess or re-sterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death. Reprocessing of medical devices intended for single use only may result in degraded performance or a loss of functionality.

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Rx only.

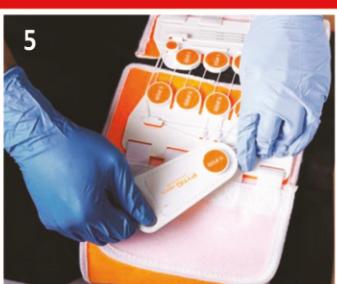
Teleflex Medical
IDA Business and
Technology Park
Dublin Road, Athlone
Co. Westmeath, Ireland

Teleflex®

www.teleflex.com

PPL-157A (2/18)

en Photos are for illustration purposes.

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en 1. Slide belt under patient and into position under the pelvis. It is recommended to remove patient clothing prior to applying T-POD.

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en 2. Trim the belt leaving a 6-8" (15-20 cm) gap over the centre of the abdomen.

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en 3. Apply Velcro-backed pulley system on each side of the gap.

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en 4. Draw the pull tab, creating simultaneous circumferential compression.

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